

ORALMEDIC®

Alleviates mouth ulcer pain in seconds!

One application of **ORALMEDIC®** relieves the pain and discomfort of a mouth ulcer in seconds. A single-use prefilled **ORALMEDIC®** swab delivers the amount of liquid material required to treat one typical mouth ulcer. It works by forming a protective barrier over the ulcer to provide optimum healing conditions for natural healing.

ORALMEDIC® is for the treatment of ulcerations in the tissue of the mouth. It is not intended or effective in the treatment of cold sores, fluid-filled blisters or vesicles in the mouth or on surface of the lips.

WARNING: Do not use if allergic to sulphur.

Instructions:

Important - Please read carefully



1. Dry affected area using the cotton drying bud.



2. Hold the applicator at arms length, away from the eyes, with coloured ring up. Gently bend the coloured ring to one side until it snaps. **ORALMEDIC®** will flow to the opposite end.



3. Apply to the affected area for 5 - 10 seconds. Use a rolling motion to completely cover the entire ulcer bed and ulcer rim. A 'stinging' sensation is experienced immediately upon application.



4. **Rinse mouth thoroughly with water and spit out.**

5. The used applicator should be disposed of after it has been thoroughly rinsed with water, or has been carefully wrapped. Dispose of swab in the bin.

An effective treatment is evident when a "white patch" is seen in and around the ulcer. This white patch may flake off during the healing period.

Note: If the ulcer pain remains after rinsing with water, it is an indication that some part of the ulcer was not covered. Repeat application immediately, one more time only, following the directions above. Thereafter do not retreat again.


Important Safety information: Do not use if allergic to materials that contain sulphur in any form. Do not use if you are pregnant, or on children under the age of 12. Avoid contact with the eyes. If eye exposure occurs, immediately remove any contact lenses, irrigate eyes for at least 15 minutes with water and seek immediate medical advice.

May cause local irritation or mild swelling when applied. If excess irritation occurs during use, rinse thoroughly with plenty of water. If condition persists, consult your doctor or dentist.

Avoid contact with clothing as it may discolour or damage.

Keep out of reach and sight of children.

Store at or below 25 °C in a dry place.

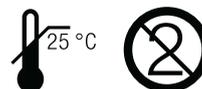

If used as directed the product cannot be reused. After the swab is activated and liquid is applied, the oral cavity is rinsed with water and spat out, with the used swab properly disposed of.

Ingredients: HybenX® (Hydroxybenzenesulfonic Acid, Hydroxymethoxybenzenesulfonic Acid, Sulphuric Acid, Water, FD&C Red 40).

This pack contains: 2 single-use applicators, each containing 0.2 ml of HybenX® solution. 2 drying buds.



EPIEN Medical, Inc. 4225 White Bear Parkway,
Ste 600, St.Paul, MN 55110 USA

HybenX® is a registered trademark of EPIEN Medical, Inc.

Distributed in South Africa by: Abex Pharmaceutica (Pty) Ltd, Suite C,
Rubenstein Ridge, 617 Rubenstein Drive, Moreleta Park, 0181

Revision date 04/2019 ORA/IFU/A